


Robotic Abdominal/Pelvic Drape

DRAPING GUIDE


This drape includes features to meet the growing needs of robotic abdominal and pelvic surgery, including a trapezoidal fenestration, clear poly instrument pouches and convenient hook-n-loop tube holders.



Step 1: Patient is in Trendelenburg position for ease of access of the robotic arm.

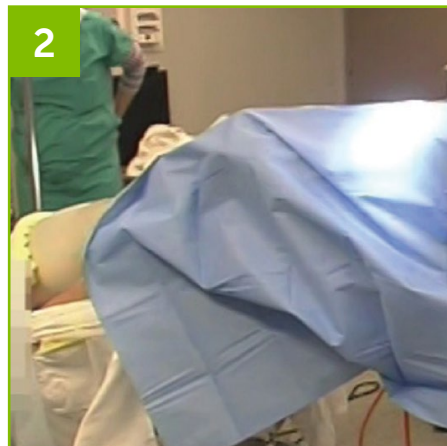

Step 2: The patients legs are draped with solid (as shown above) or clear leggings.

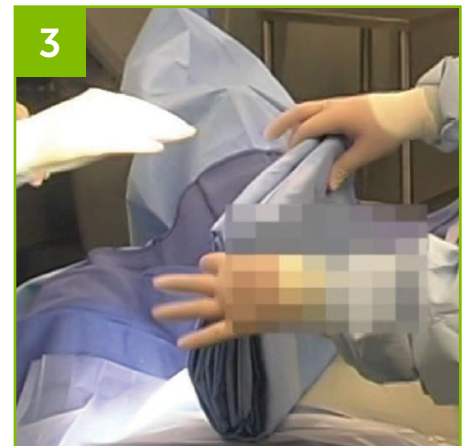

Step 3: Position the drape on the lower abdomen and unfold laterally.



Step 4: With one person on either side of the patient, unfold toward the patient's feet, making sure you position the rounded portion over the patient's legs. Unfold the remainder of the drape toward anesthesia and secure to IV poles.



Step 5: Remove the release liner and secure the fenestration to patient's abdomen.



Step 6: Grasp the flaps and secure to the leggings with a towel clip to ensure proper coverage.

Featured drape (29240/29241NS)

For more information for clinicians and product code ordering information, please visit:
www.halyardhealth.com

Call 1-844-HALYARD (1-844-425-9273)
 in the United States and Canada.